

Traditional Model Guidelines

Sending a model:

- Carefully package and ship your articulated upper and lower casts that have been thoroughly disinfected.


If needed, Ivoclar Vivadent can provide you a container to ship the model.

- Be sure the operative cast has removable, clearly trimmed dies with clearly marked margins
- All dies must be ditched for accurate scanning and easy identification of margins
- Diagnostic wax-ups may help facilitate the design but are NOT essential



Model workflow:

When sending a model to Ivoclar Vivadent you will receive a design of your restoration to approve. Once you approve the design the restoration will be milled and sent back to your laboratory ready for final characterization.

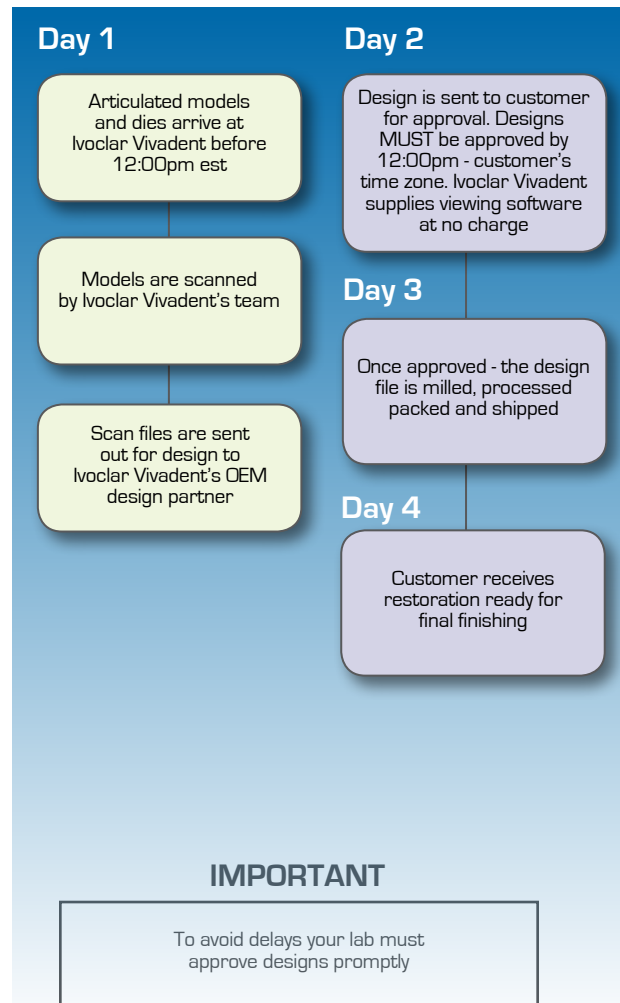

Please do not send:

- Broken casts, chipped models or dies with faulty or loose dowel pins



- Wax bites or impressions - we only need the articulated casts with removable, clearly trimmed and marked dies that have been disinfected thoroughly
- Models with an incomplete or no script/work order (to avoid delays)

1-877-934-2336
www.ivoclarvivadent.com

ivoclar vivadent
passion vision innovation

Laboratory _____
Contact _____
Address _____
Phone _____ email _____
Case ID _____

Restoration Type:
 Crown Bridge
 Coping Framework

Material Type:
 IPS e.max[®] CAD
 Translucency (single one): HT MT LT MO
 Zenostar[®] MT (crowns & bridges)
 Zenostar[®] T (crowns & bridges)
 Zenostar[®] MQ (framework)
 Tetric[®] CAD (temporary crowns & bridges)
 Shade Options: A1 A2 A3 A3.5 B1 B3

Shade: _____

Notes: _____

United States
 Ivoclar Vivadent, Inc.
 10100 Highway 61
 Troy, Michigan 48063
 1-877-934-2336

Ivoclar Vivadent, Inc. is a registered supplier of dental products in the United States. All other trademarks and registered trademarks are the property of their respective owners. © 2014 Ivoclar Vivadent, Inc.

Ivoclar Vivadent can provide you with work order pads.